


Standard Line Measuring and Sizing chart

Sizing Rules

Be sure measurements are accurate. When on borderline between sizes, consider amount of edema present & anticipated reduction. If significant edema is present and reduction is anticipated, select smaller size; if edema is minimal, select larger size. If measurements do **NOT** fall within **any [one] size category**, please see the custom sizing chart.

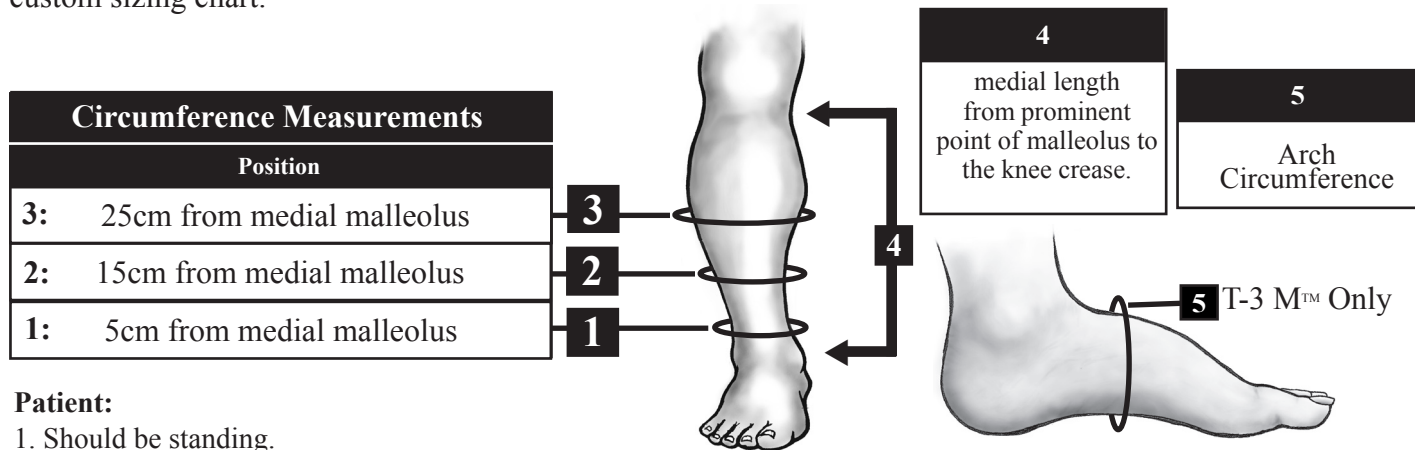

Circumference Measurements	
Position	
3:	25cm from medial malleolus
2:	15cm from medial malleolus
1:	5cm from medial malleolus

Patient:

- Should be standing.
- Limb should be at its smallest and limb size at time of measurements should be maintained to assure a proper fit of garment.
- Be sure to select a legging size and length for all product types; and, a foot size for T-3 M™ only.

	Measurements		Small		Medium		Large		XL	
	Left	Right								
1 5cm			19 - 22cm	22 - 27cm	25 - 32cm	30 - 37cm				
2 15cm			21 - 29cm	28 - 36cm	34 - 42cm	40 - 48cm				
3 25cm			25 - 36cm	33 - 44cm	41 - 51cm	48 - 58cm				
4 Length			Under 36cm Short	Over 36cm Long	Under 36cm Short	Over 36cm Long	Under 36cm Short	Over 36cm Long	Over 36cm Long	
5 Arch Circumference (Foot Size)			19-26cm Small	24-31cm Med	19-26cm Small	24-31cm Med	19-26cm Small	24-31cm Med	19-26cm Small	24-31cm Med

(rev.0109) ©2009 CircAid Medical Products, Inc. All rights reserved.

Foot Piece (Ankle-Foot Wrap)

Each T-3 M includes an attached closed heel Juxta-Lite™ ankle-foot piece. The foot piece can accommodate the circumference ranges listed above (#5), and a foot length of 16cm or greater. The T-3 EZ™ includes an attached EZ single-band ankle-foot wrap and the T-3 Boot™ includes an attached CircPlus™ II ankle-foot wrap. Both of these foot pieces are adjustable and fit most. The C-3 EZ™ and C-3 Wrap™ do not include a foot piece and one must be ordered separately. If foot measurements fall outside of these ranges and you fit into a standard size legging please contact us for additional assistance with your order.